# FIXIERBANDAGE STANDARD, 60X10 CM

## ANLEITUNG

Art.-Nr.  21910 · Kategorie:  Bandagen

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## ENGLISH

### DESCRIPTION AND INDICATIONS

The electrodes in conductive silicone have been designed for electro-stimulation use in physiotherapy and beauty-care.

### CONDUCTIVE SILICONE ELECTRODES

Available in various shapes and sizes. These models are available with 2 or 4 mm plug or socket connectors. All models are also produced with a 15cm cable and 4mm socket. These electrodes are mainly used in the physiotherapy field.

### SPONTEX ELECTRODES HOLDING BAGS

### SYNTHETIC BUCKSKIN ELECTRODES HOLDING BAGS

Hydratable holding bags available in various sizes, to be used with the above conductive silicone electrodes.

### REUSABLE CONDUCTIVE SILICONE BANDS

#### MODELS PG901 PG902

Conductive silicone bands available in rolls, in 25 or 50 mm widths, for electro-sculpture treatment in beauty-care. For use with cables fitted with alligator-clip connectors.

#### MODELS PG901/2 PG906/2

Conductive silicone band available in rolls, in 50 or 100 mm widths, with 2mm central channel. Having cut the band to the desired length, the plug at the end of the cable of the device is then inserted into this channel. Particularly suitable for use with TENS equipment, it enables the user to personalise the dimension of the electrodes as required. To be used with cables fitted with 2mm plug.

### HYDRO COMPRESSED SPONGES

Hydrocompressed or Spontex sponge sheets in different dimensions, to be used with the electrodes in conductive silicone as above or for vacuum applications

### DIRECTIONS FOR USE

Having identified the application area, follow the directions given below for the chosen technique.

You can follow two different techniques depending on whether electrodes or bands are used:

### USE CONDUCTIVE SILICONE ELECTRODES

- Distribute the electro-conductive gel on the flat surface of the electrode which is to be placed in contact with the skin.
- Secure the electrode using the special band, following the manufacturer's instructions.
- Connect the electrode to the cable of the stimulator.
- After the session, switch off the stimulator, disconnect the cable from the electrode and remove electrode and band.

### USE WITH HYDRATED HOLDING BAG AND/OR SPONGE

- Select the suitable bag or sponge sheet or spontex bag/sheet to be used with the electrode.
- Moisten the band with water or medication according to the type of application.
- Connect the electrode to the cable of the electro-stimulator.
- Insert the electrode inside the bag or on the sponge or Spontex sheet and fix everything with the suitable elastic band following the instructions of the band manufacturer
- After the procedure, switch off the stimulator, disconnect the electrode from the cable and remove the band, the sponge and the electrode.

### USE REUSABLE CONDUCTIVE SILICONE BANDS

#### USE OF BANDS SERIES PG901 AND PG902

- Before securing the conductive silicone band to the skin, apply a moist bandage.
- Apply the band and then secure it by overlapping the two edges and fastening them with alligator clips.
- To avoid venous stasis, do not over-tighten the band.
- Use OF MODELS PG901/2 PG906/2
- Cut the band to the desired length.
- Use the product, following one of the two techniques described above (gel or hydrated holding bag).

### WARNINGS

- The product supplied is not sterile.
- The equipment and the connecting leads used with this device must all be compliant with the regulations in force.

**N.B. Only ever use equipment designed for electro-stimulation at low current intensity.**

- The device must be connected and started up by qualified personnel.

- Do not apply electrodes or bands on skin affected by abrasions, erythema or wounds.

- Hydro-compressed sponges fear humidity.

- Strong mechanical stresses or rubbing can damage the product.

### CLEANING

Detergent solutions (non-ionic surface-actives) and disinfectant solutions (components with quaternary ammonium of 0.1 - 0.2%) must be used for the cleaning of the products. For the use of these products follow the relative instructions. Only wash in cold water (30° max). After cleaning rinse under running water.

### PRODUCT LIFE

The life of the product is a consequence of the number of uses and the state of maintenance and conservation of the product itself. As soon as deterioration of materials or component parts is observed, the product must be discarded.

### STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

### GENERAL NOTES

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

### WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force.

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**Sport-Tec GmbH**