FERROX TILT TABLE RAFFAELLO

ANLEITUNG

Art.-Nr.  22975 · Kategorie:  Kipptische

Art.250174D

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TIPS TO BE OBSERVED TO USE THE TILT TABLE SAFELY

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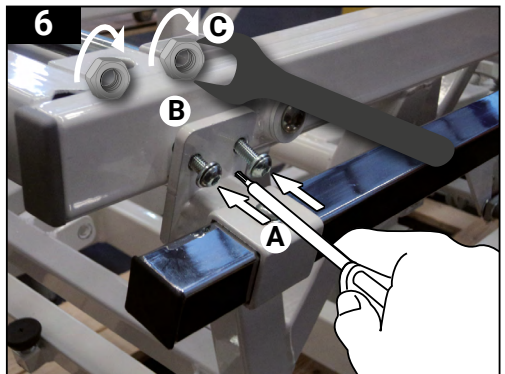
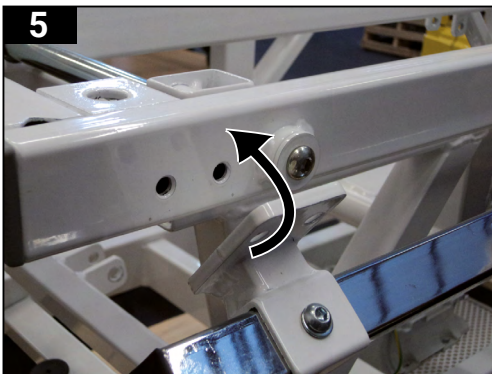
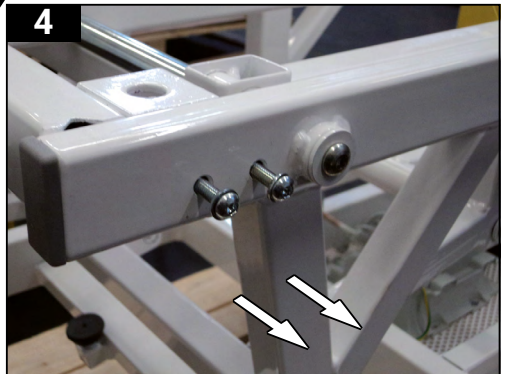
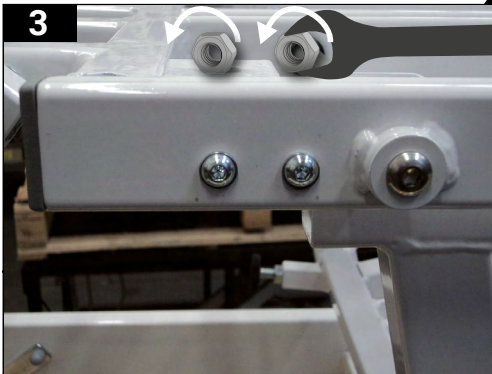
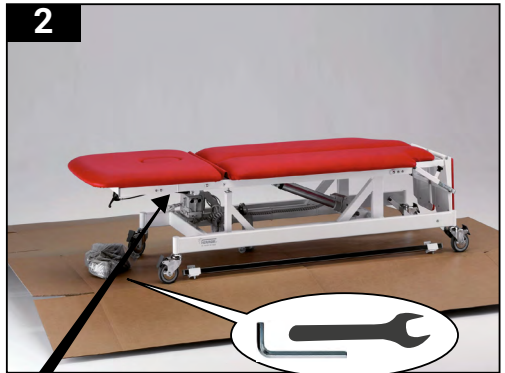
As part of ongoing development program, Ferrox srl reserves the right to alter or modify the specifications and materials without prior notice.

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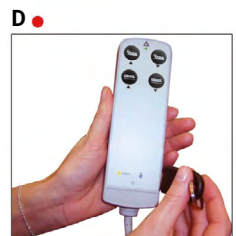
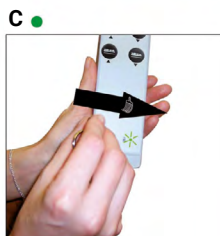
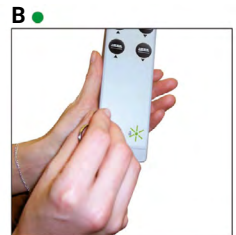
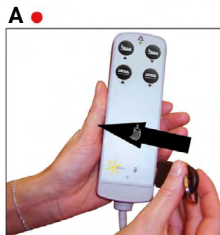
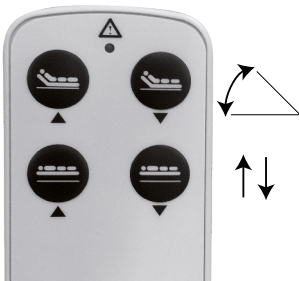
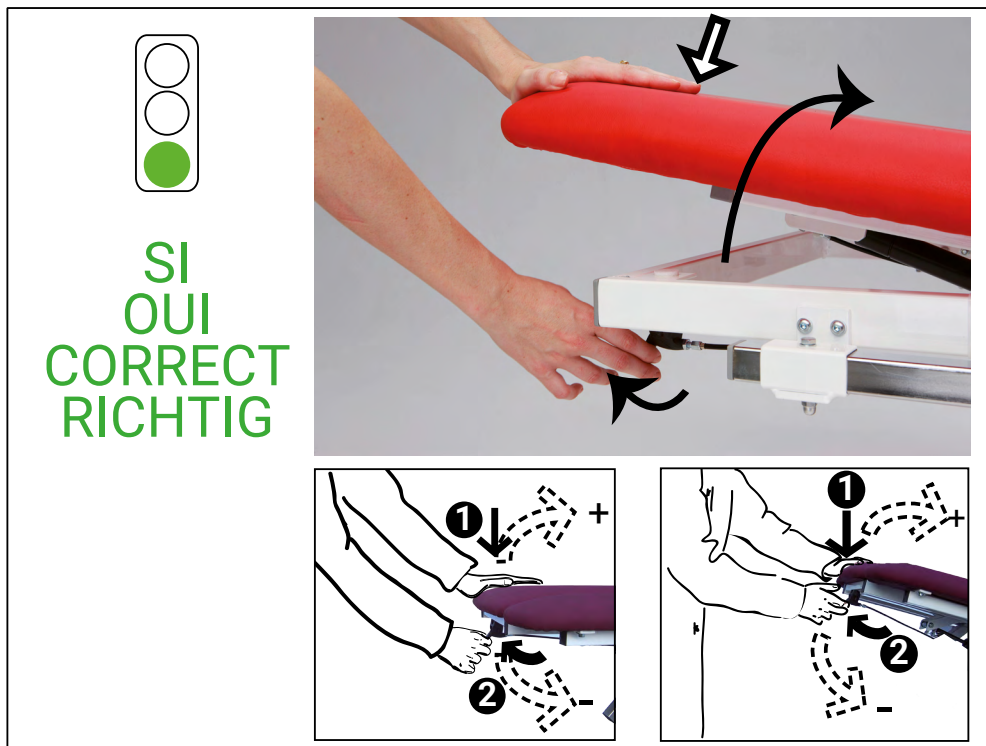
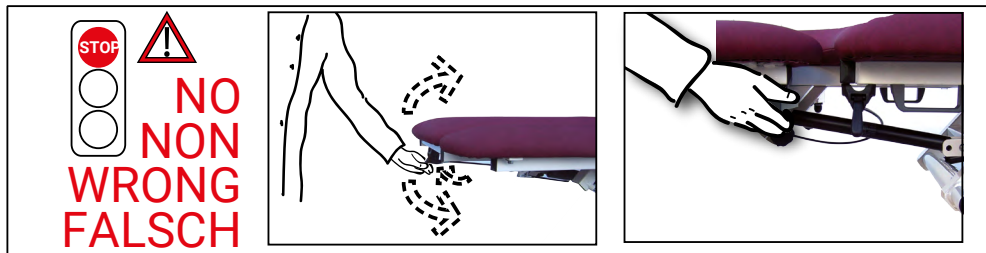
To avoid injury of the practitioner and the patient, hold and tilt the concerned table section with one hand before activating the lever for the gas-spring assisted tilt regulation, to avoid the section to spring back into its original position.
 Operate the treatment table only with the patient lying down.

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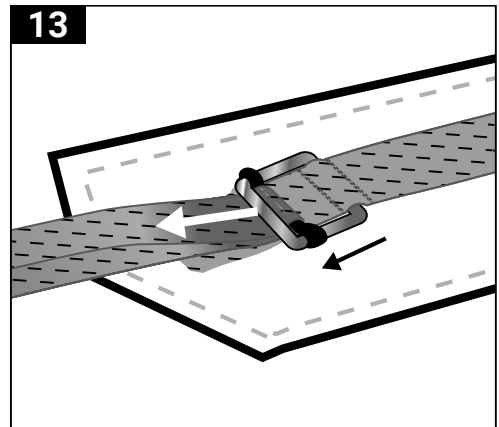
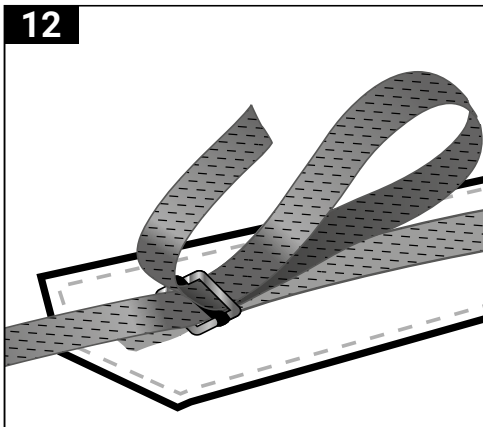
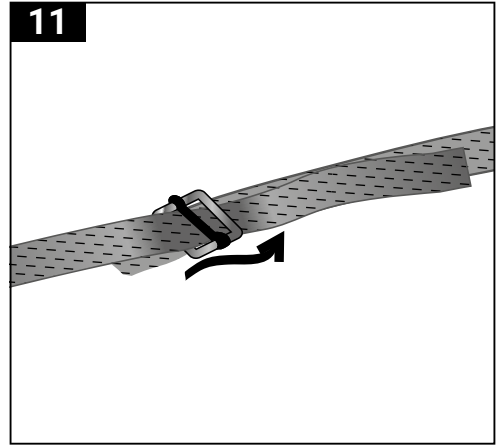
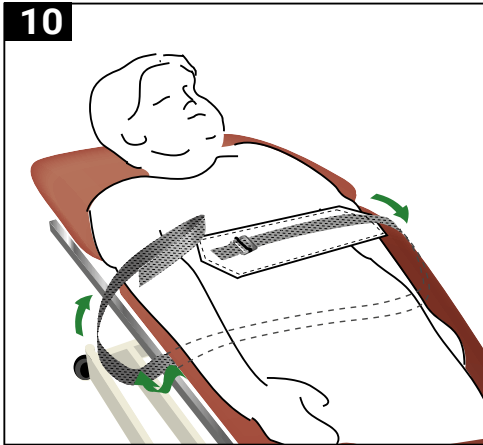
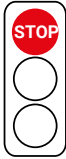
ALWAYS immobilize the patient with the straps provided. Make sure that the straps are in their correct position and fastened and that the patient is fully secured before operating the tilt facility.

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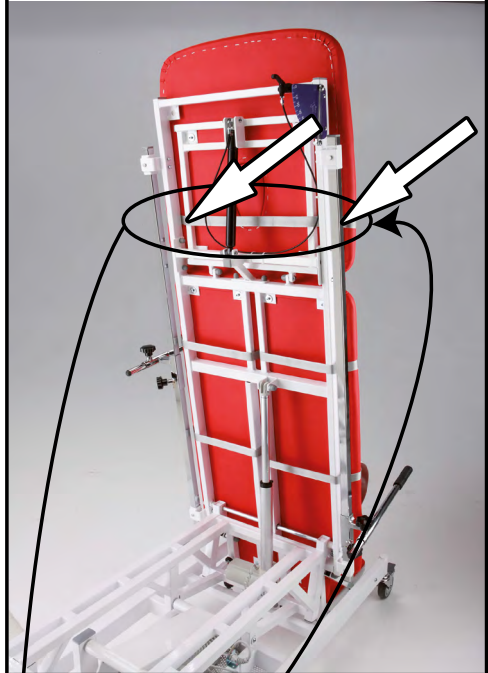
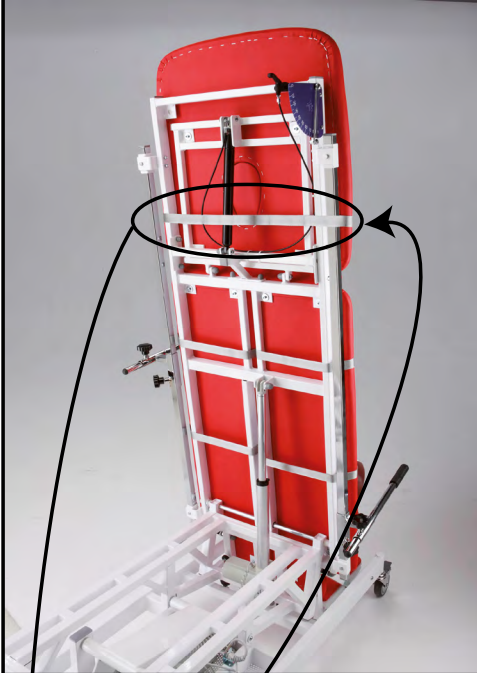
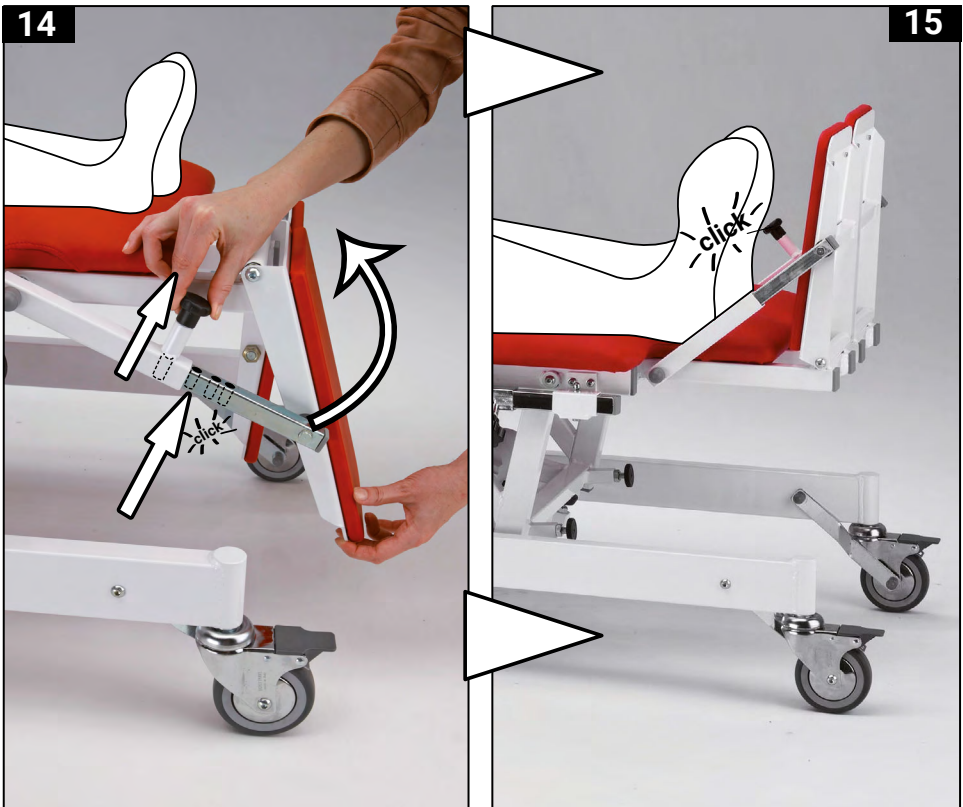
 It is MANDATORY that the patient is lying down BEFORE the footrest angles are adjusted Make sure that the footboard is locked and the release knob is fully inserted.

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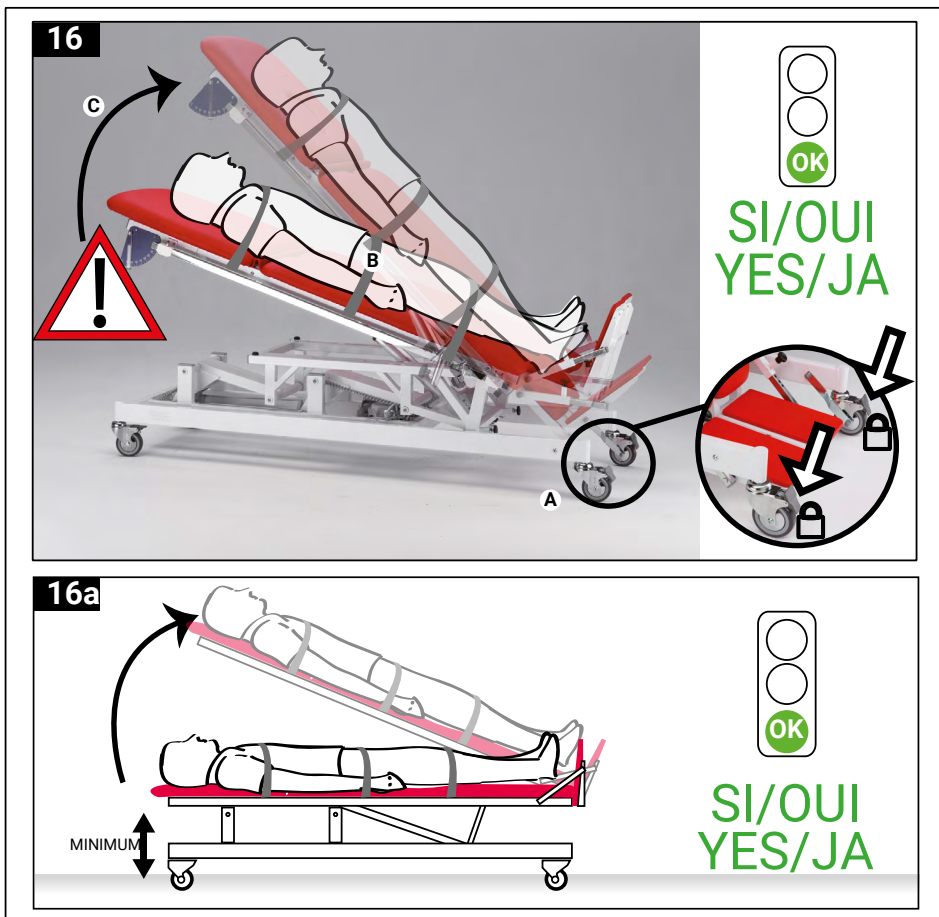
BEFORE bringing the patient in an upright position, it is MANDATORY that:

- the two front wheels are locked (16)
- the resting surface is at its lowest possible height-adjustment position (16a)
- the resting surface is in horizontal position (16a)
- where present, the backrest is in horizontal position
- the two footrests are facing upward and outward but NEVER downward (17)
- the patient is safely secured on the treatment table with the straps provided.

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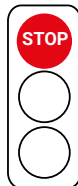
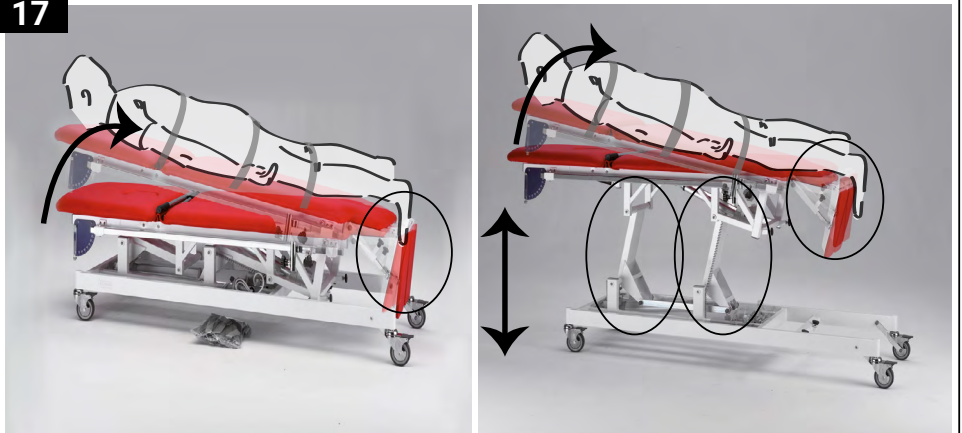
NEVER bring the treatment table into vertical position with the footrests facing downward -- they must for this purpose ALWAYS be in the upward position

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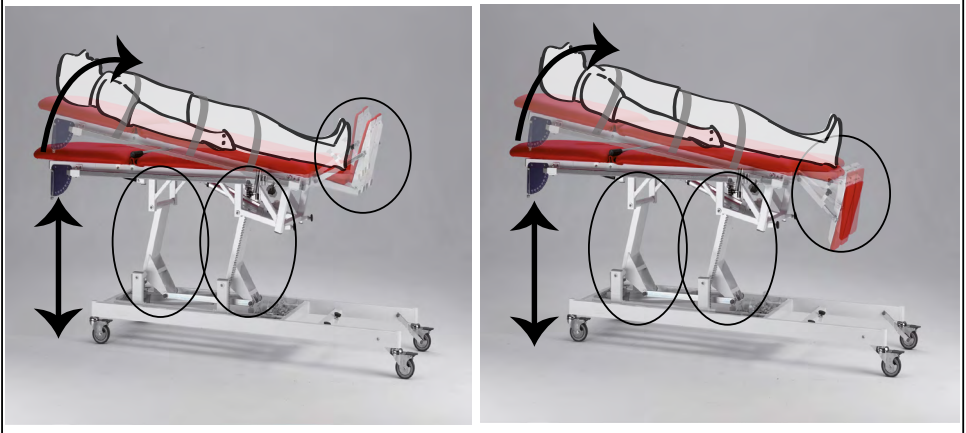
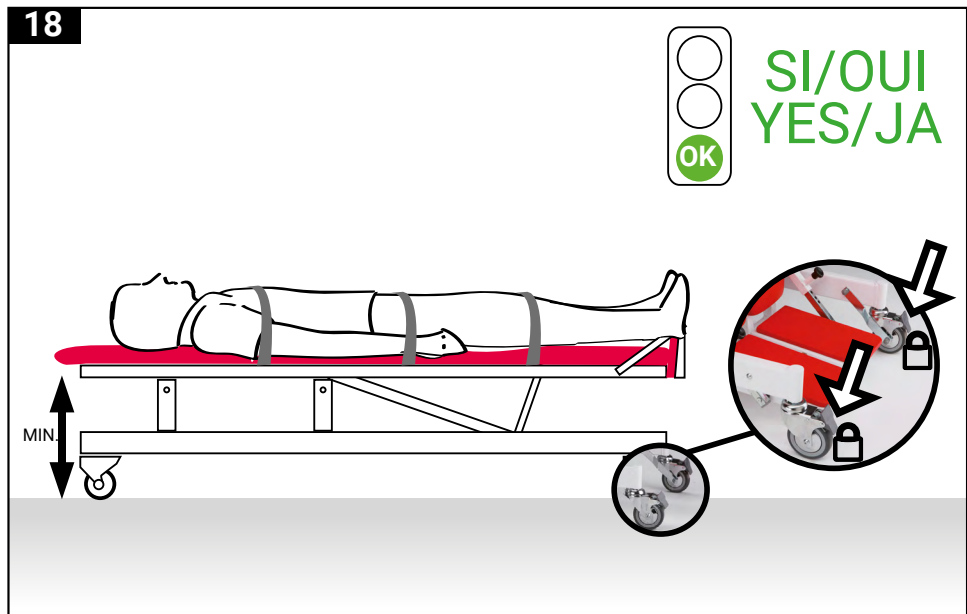
 **PRIOR to transferring or moving the patient, it is MANDATORY that the resting surface of the tilting table is:**

- in a horizontal position (18)
- at its lowest possible height-adjustment position
- make sure wheels are locked

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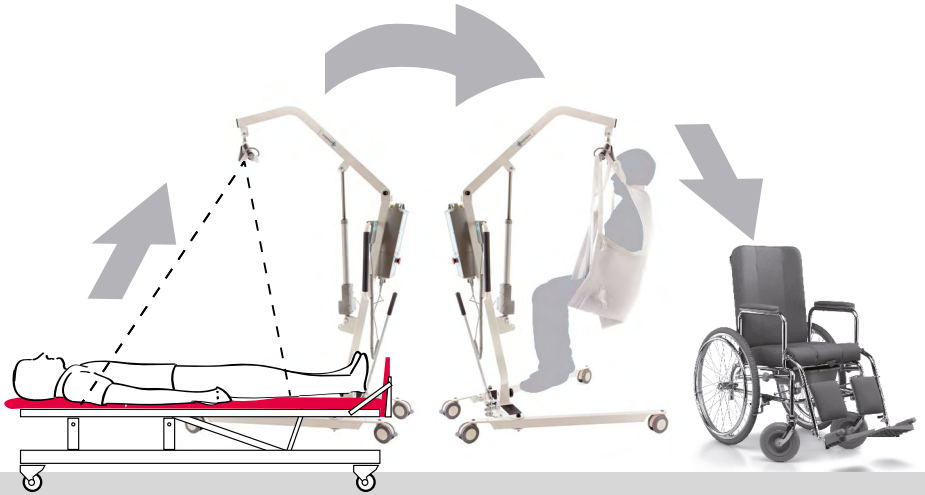
 This couch is not designed for patient transportation.

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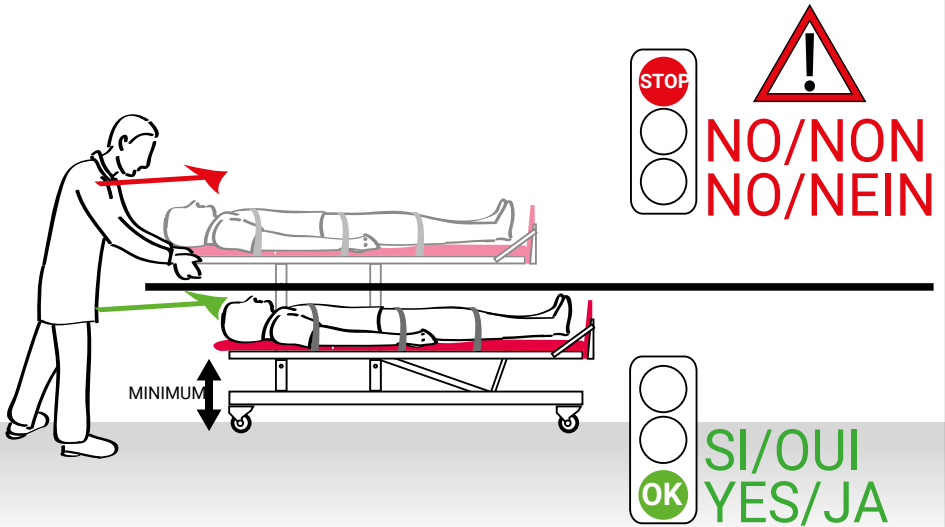
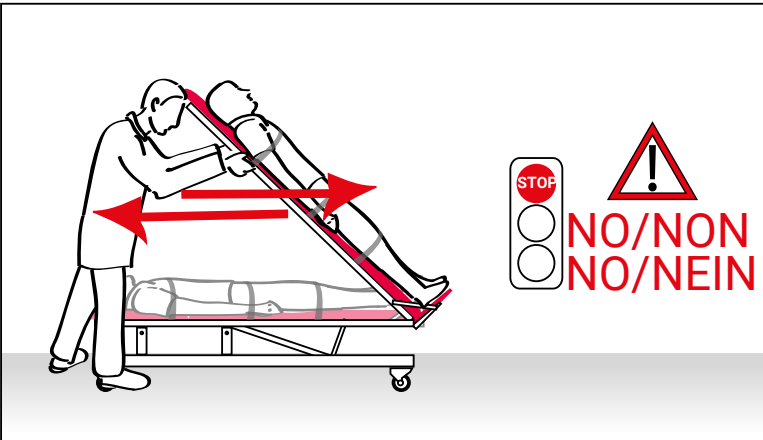
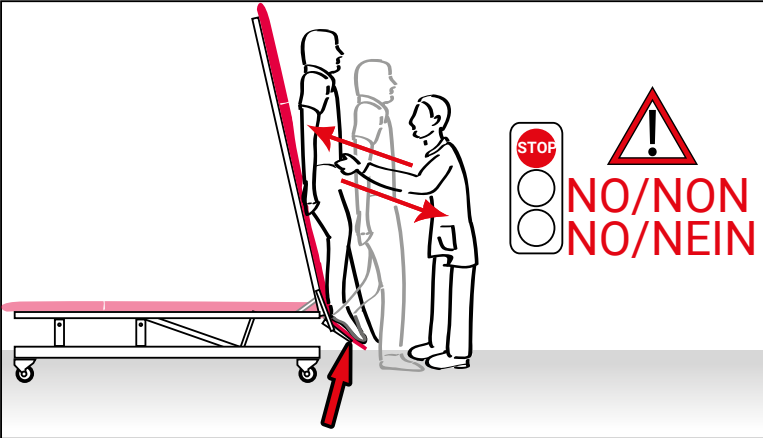
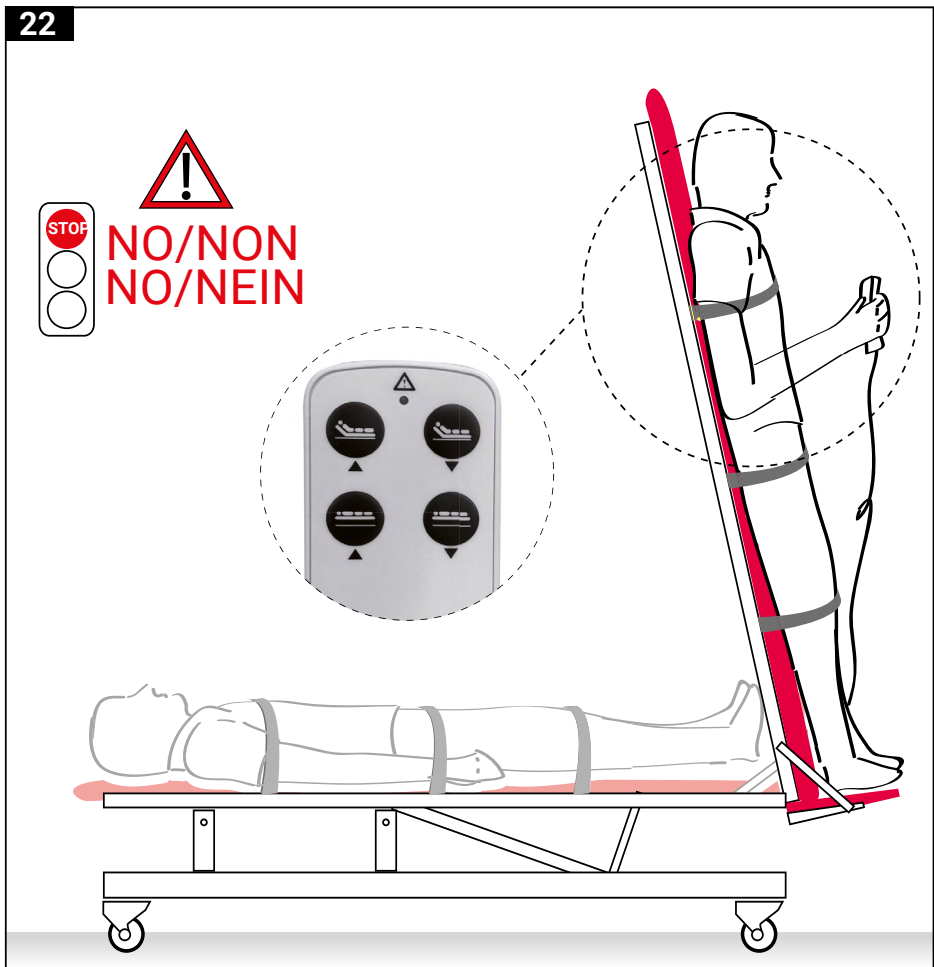
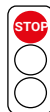
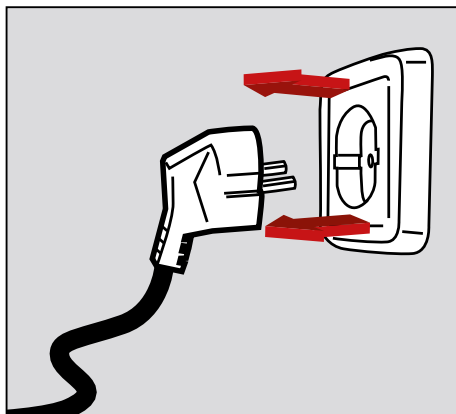
 **ALWAYS** ensure that no objects on or around the table would compromise or impede the free moving of the table and its parts.
ALWAYS be very careful while adjusting the tilt/height of the treatment table. The user must keep all limbs at a safety distance from the moving parts during any electronically or mechanically effected adjustment of the table position.

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 Disconnect the electric lead prior to re-locating the couch and ensure the lead is not trapped in the mechanism or in a position to cause a trip hazard.

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 **NEVER** modify the equipment. Any kind of unauthorised manipulation of the device may impair its proper functioning.

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Sport-Tec GmbH