DE 

EN 

FR 

IT 

ES 

NL 

## **SPORT-TEC MOBILER MASSAGESTUHL RELAX**

SPORT-TEC MOBILE MASSAGE CHAIR RELAX

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Ihr Sport-Tec Team

## WELCOME TO SPORT-TEC

Thank you for your purchase. We are pleased that you have opted for a Sport-Tec product and hope that you really enjoy using it. In this manual, you will find all the important information for use. If you have any further questions, please contact us – we will be happy to help you.

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**S. 4-9**

## EN - MANUAL

Read this manual carefully before use. Follow the instructions, and keep this user manual in a safe place.

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
# GENERAL INFORMATION

The massage chair impresses with its stable yet compact design. It can be folded up to save space and is easy to transport. The adjustable components allow ergonomic adjustment to different body sizes and ensure comfortable positioning during use. The easy-care surface can be cleaned hygienically. The massage chair is particularly practical for mobile use thanks to its flexible handling.

## SCOPE OF DELIVERY

| Designation                | Quantity |
|----------------------------|----------|
| Mobile Massage Chair Relax | 1x       |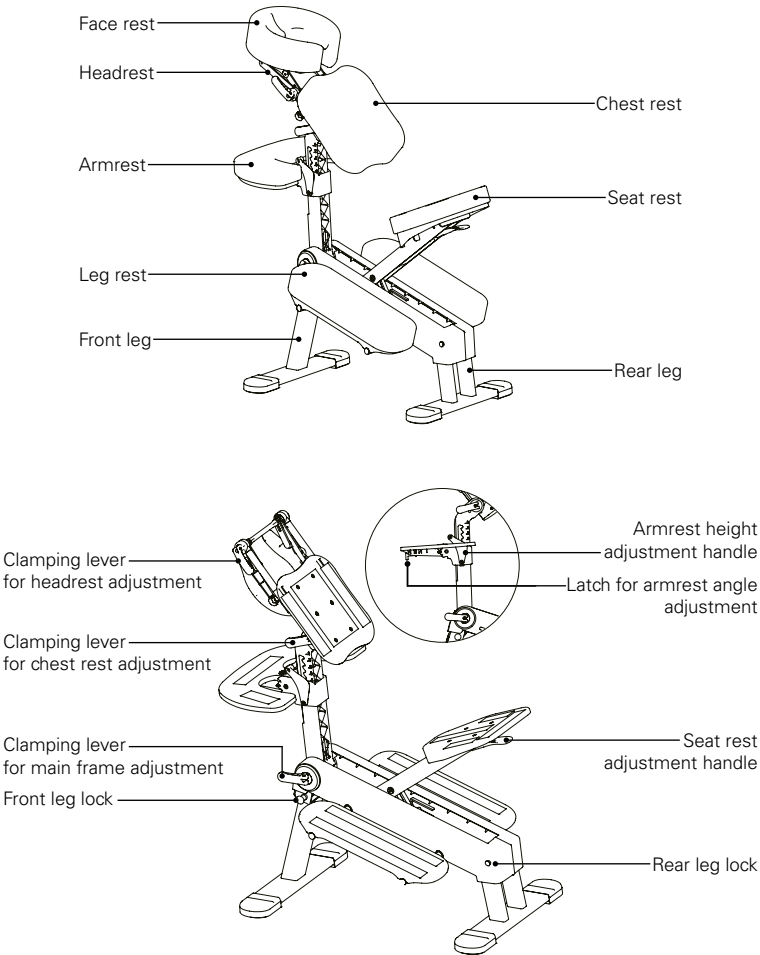

# SAFETY INSTRUCTIONS

Improper use can lead to injuries.

Before each use, check the product for damage, and to ensure that all parts are correctly fastened.

Only use the product on a safe and stable surface.

The maximum load capacity of the main frame of 204 kg (or 35 kg for armrest and headrest) must not be exceeded.

This product is not a toy. Always keep children away from the product and accessories.

Never leave children unsupervised in a room with the product.

Never carry out repairs independently.

Only have repairs carried out by the manufacturer or the authorised customer service team.

It is strictly forbidden to dismantle the massage chair.

Regularly check the stability of the assembled massage chair.

Do not touch the massage chair with sharp or pointed objects to avoid damage.

Avoid collisions when moving or transporting the device.

## ASSEMBLY

### **Important information:**

Keep this assembly manual in a safe place for future reference. If you move home or pass the product onto someone else, make sure that this manual remains with the product.

## PREPARATION

Read through all the work steps carefully before starting assembly.

First, check the condition of all parts and ensure they are complete. Do not use damaged parts. If any parts are damaged or missing, please contact our customer service team.

## WORKING ENVIRONMENT

To assemble the product, choose a clean, flat surface with sufficient space and good lighting.

Ensure the surface is non-slip to avoid scratching the floor and the product.

To avoid personal injury, keep children and pets away from the work area during assembly.

DE

EN


FR

IT

ES


NL

## ASSEMBLE MASSAGE CHAIR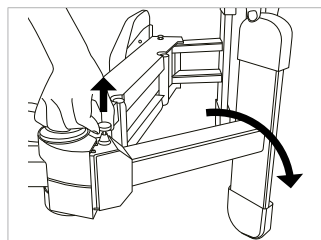


### Step 1: Fold out the front leg

Pull the locking mechanism upwards.  
Fold the leg outwards until it engages.  
Release the lock and check that the leg is securely locked in place.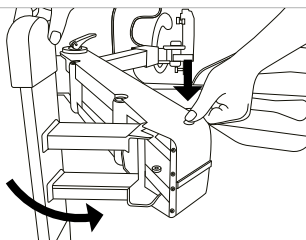


### Step 2: Fold out the rear leg

Press the locking mechanism down.  
Fold the leg outwards until it engages.  
Release the lock and check that the leg is securely engaged.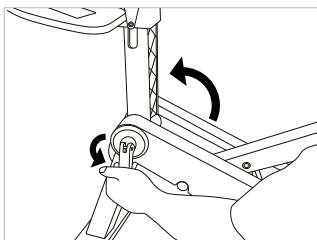


### Step 3: Unfold the main frame

Release the clamping lever.  
Fold the main frame upwards.  
Close the clamping lever and check that the main frame is securely fixed.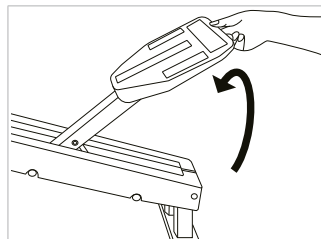


### Step 4: Unfolding the seat

Raise the seat as far as it will go and then lower it again until the linkage engages.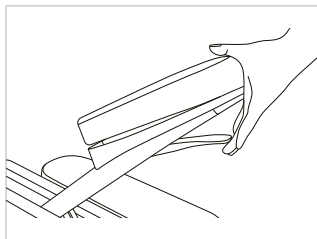


### Step 5: Adjusting the seat height

Pull up the handle to adjust the seat cushion.  
Depending on the desired height, engage in the appropriate holder.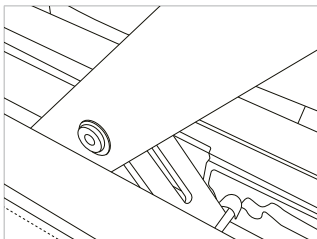


### Step 6: Adjusting the height of the armrests

Press down the handle to adjust the height of the armrests.  
Adjust the armrests to the desired height.  
Release the handle.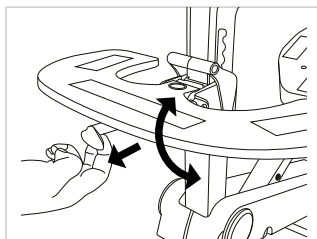


### Step 7: Adjusting the angle of the armrests

Pull out the locking mechanism.  
Set the desired angle of inclination for the armrest.  
Release the lock so that the armrest engages.  
Position the upholstered section with the Velcro fastener on the intended surface and press firmly into place.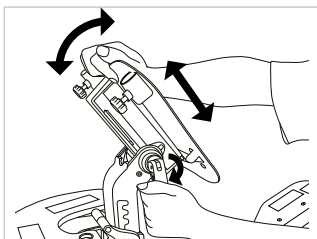

### Step 8: Adjust the position of the chest pad


Release the clamping lever.  
Adjust the chest section to the desired height.  
Adjust the required angle by tilting the chest section.  
Close the clamping lever and check that the chest section is securely fastened.  
Position the upholstered section with the Velcro fastener on the intended surface and press firmly into place.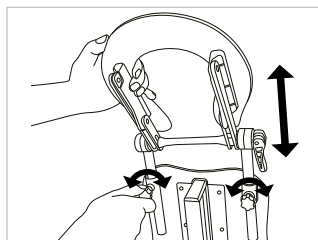

**Step 9:  
Fitting the headrest**

Insert the headrest rods into the guide sleeves on the chest section.

Adjust the headrest to the desired position and tighten the twist lock.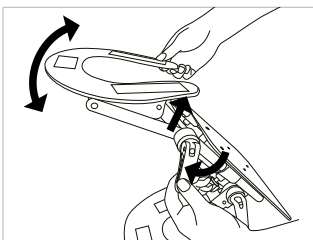


**Step 10:  
Setting the headrest angle**

Loosen the clamping lever

Set the required angle by tilting the headrest.

Close the clamping lever and check that the headrest is securely fixed.

Position the upholstered section with the Velcro fastener on the intended surface and press firmly into place.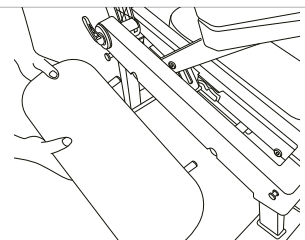

**Step 11:  
Fitting the leg pads**

Insert the leg pads into the main frame.


Position the upholstered part with the Velcro fastener on the intended surface and press firmly into place.

The leg pads can be removed if necessary. Simply pull out the leg pads.

## FOLDING MESSAGE CHAIR

**Important information:**

Remove all padded parts before folding and store them safely.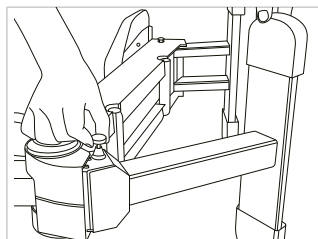


**Step 1:  
Folding in the front leg**

Pull the locking mechanism upwards.

Fold the leg inwards.

Release the lock and ensure that the leg can no longer be moved.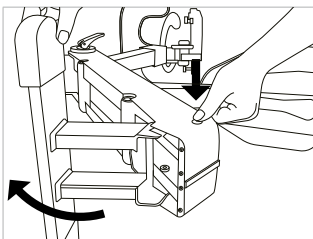


**Step 2:  
Folding in the rear leg**

Press the locking mechanism down.

Fold the leg inwards.

Release the lock and ensure that the leg can no longer be moved.

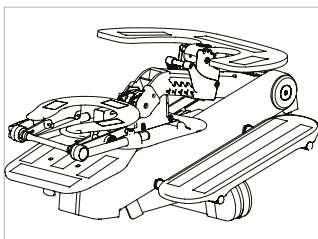**Step 3:  
Folding in the chair**

Fold in all parts in reverse order of assembly.

## NOTES ON USE

The device may not be used by persons under the age of 18 for the intended area of application.

The user must have sufficient knowledge to use the massage chair. Special care is required for customers with physical, sensory or mental disabilities. This is the responsibility of the user.

## CARE AND CLEANING

An ambient temperature that is too high or too low can accelerate the ageing of the material and cause it to discolour, deform, harden, become brittle and even detach from the surface.

Clean the product after use with a soft cloth and mild detergent, alcohol-free disinfectant or water. Wipe with a dry cloth.

Do not use any aggressive, alcohol-based or abrasive cleaning agents, as these can damage the material.

Make sure that the massage chair is completely dry after cleaning.

Store the massage chair in cool, clean and dry conditions.

We recommend covering the upholstered parts when the massage chair is not in use to protect them.

## DISPOSAL

Dispose of the product in accordance with local regulations.

For larger items, please use the bulky waste service or take the product to a recycling centre.

If possible, disassemble the product into its individual parts to enable optimal recycling.

Dispose of the packaging materials in an environmentally friendly manner in accordance with local regulations.

Separate packaging materials (cardboard, plastic, etc.) and recycle them accordingly.

# PRODUCT FEATURES

|                                | Sport-Tec Mobile Massage Chair Relax                      |
|--------------------------------|-----------------------------------------------------------|
| Part. no.                      | 23790                                                     |
| Colour                         | 01-blue, 18-anthracite, 19-orange, 21-lime                |
| Max. load capacity             | 204 kg (main frame) or 35 kg (armrest and headrest)       |
| Weight                         | 13 kg                                                     |
| Dimensions when folded (LxWxH) | 78.5x48x37.5 cm                                           |
| Dimensions (LxWxH)             | 80-87x50x112-125 cm                                       |
| Material                       | Metal and plastic (frame), imitation leather (upholstery) |
| Country of origin              | Made in China                                             |

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**Sport-Tec GmbH**