# NUBIS<sup>®</sup> PRO

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# NUBIS®

## CONTENT

1.	Introduction	3
2.	Product overview	3
3.	Assembly / installation	4
4.	Disassembly	5
5.	Operation	5
6.	Safety instructions	6
7.	Cleaning and maintenance	7
8.	Technical data	7
9.	Accessories and Spare parts	8
10.	Warranty and Liability	8

# NUBIS®

## 1. INTRODUCTION

Thank you for choosing NubisPro portable massage table.

NubisPro is portable massage table. Table is created for masseurs, especially for their use in the field, and at their privat practices, especially where having space problems.

Table is very stable, firm and ready for firmest massage techniques.




It's easy to set up, when not in use it can be easily transported in a backpack with wheels.

In order to use NubisPro correctly and safely, it is important to read these User manual carefully before starting to use the product.

## 2. PRODUCT OVERVIEW

NubisPro consists of:

1. Mattress
2. Head rest with breathing hole
3. Table base (frame and legs)
4. Electric pump
5. Backpack with wheels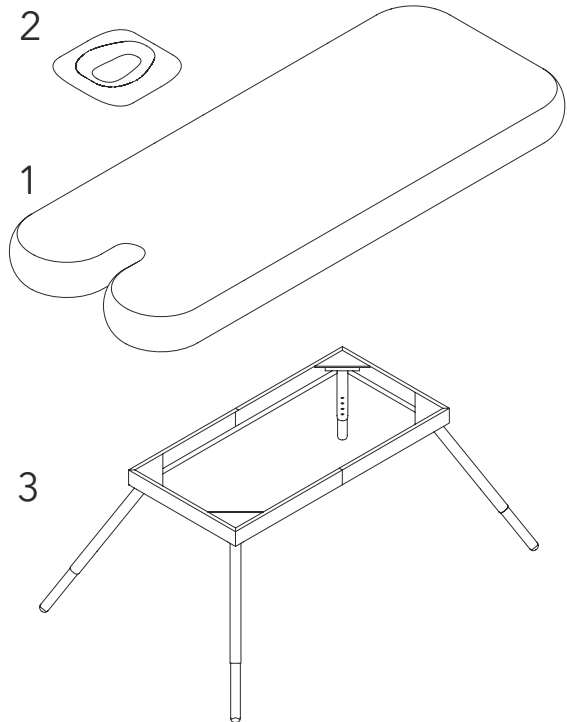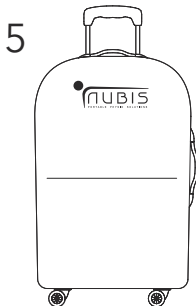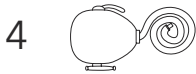


## 3. ASSEMBLY/ INSTALLATION

3.1. Take the NubisPro outside the cardboard packaging and check everything for possible transport damage.

3.2. Read this manual carefully before using the equipment.


3.3. Open the backpack and take out in following order:

1. Frame
2. Legs
3. Mattress
4. Electric pump
5. Head rest

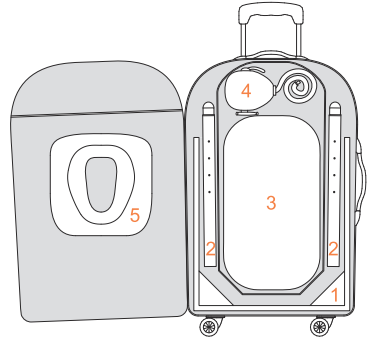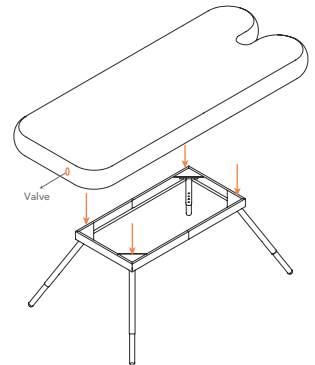
3.4. Base assembly:

1. Open the frame and put it on the floor.
2. Insert the legs into the frame, the button clip has to be engaged into the pinhole.
3. Adjust the height of the table you want to work on, by pulling out the inner part of the leg and lock it with the button clip to the selected pinhole on each leg.
4. Turn around the assembled base and place it on the floor.

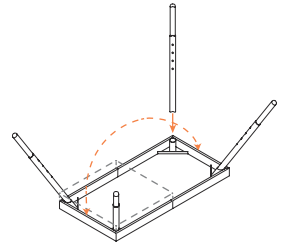
3.5. Mattress inflation:

1. Place the mattress on the frame.
2. Unscrew the valve cap, push button into the "OUT" position.
3. Take the electric pump.
4.  Prior to connection to the mains supply, check that the voltage corresponds with that written on the plate.
5. Fit the black adaptor of the tube on the inflation side of the inflator (OUT).
6. Set the pump nozzle to the mattress valve.
7. Check that the ON/OFF switch is set to (0).
8. Plug the cable into the mains supply and turn the switch to the (I) position, the inflation will start. It takes approx. 35 seconds to inflate.
9. When the maximum pressure is reached, switch off the inflator and take out the tube.
10. After inflation tighten the valve cap (clockwise). Only the valve's cap can ensure final air tightness.

3.6. Mattress and head rest attachment:

1. Attach the mattress precisely to the designated area on one side of the frame, using Velcro straps.
2. Do the attachment on the other side of the frame.
3. Attach the head rest using Velcro straps.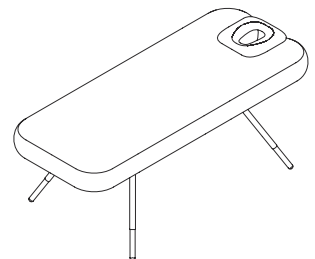

## 4. DISASSEMBLY


4.1. Remove head rest and unfasten Velcro straps from the side of the mattress.

4.2. Mattress deflation:

1. Unscrew the valve cap, push the button into the "IN" position.
2. Take the electric pump.
3. Fit the black adaptor on the deflation side of the inflator (IN).
4. Set the pump nozzle to the mattress valve.
5. Check that the ON/OFF switch is set to (0).
6. Plug the cable into the mains supply and turn the switch to the (I) position, the deflation will start.
7. When the mattress is completely deflated, switch off the inflator and take out the tube. Tighten back the valve cap.
8. Place the mattress on the floor, precisely fold and wrap it back into initial position.


4.3. Base disassembly:

1. Turn around the assembled base and place it on the floor.
2. Remove the legs from the frame and fold them into initial position.
3. Close the frame back into initial position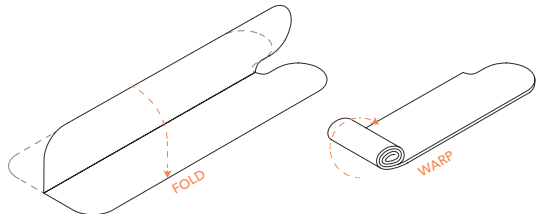

4.4. Insert into the backpack in following order:

1. Head rest
2. Electric pump
3. Mattress
4. Legs
5. Frame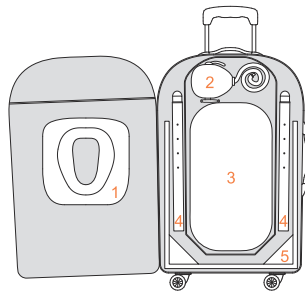

## 5. OPERATION

5.1. Head rest adjustment:

Head rest is adjusted with Velcro straps and can be moved for 15 cm upwards, recommended for taller persons.

5.2. Height adjustment:

You can adjust the table to optimum working height by adjusting the legs.

For the adjustment procedure please refer to section 3. 4. 3 (Base assembly).

5.3. Air pressure mattress adjustment:

You can adjust the hardness of the mattress by adjusting the air pressure.

You can easily release the air pressure by pushing the button into the "IN" position.

You can increase back the air pressure according to the procedure in section 3. 5. (Mattress inflation).



## 6. SAFETY INSTRUCTIONS

### 6.1. General instructions:

- The product is meant to be used for massage.
- Keep children away from equipment when no supervision is at hand.
- Use a clean couch cover for every individual, in order to guarantee a hygienic use.

### 6.2. Table handling:

- Place the table on a flat surface.
- Do not sit on the head section or foot end.
- The maximum working load of 250 kg should not be exceeded.
- Never adjust the height of the table with a person.

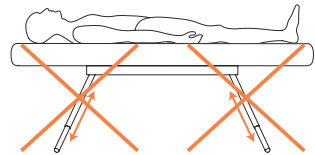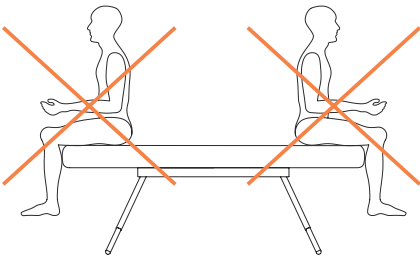
### 6.3. Mattress handling:

- Do not expose inflated mattress to extreme temperature conditions (allowed limit from -20°C to +50°C).
- Prevent damage to the upholstery from sharp objects.
- Do not fold and store the mattress, when wet. After cleaning or disinfecting, to prevent damages, allow material to dry properly before usage or storage.
- Follow carefully the cleaning and maintenance instructions of upholstery (Section 7.), in order to prevent damage of the upholstery.

### 6.4. Frame installation:

- When opening and closing the frame, be careful not be trapped between the joints. Follow the instructions carefully!

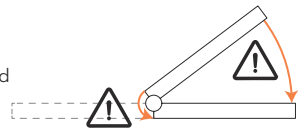### 6.5. Frame installation:

- Prior to connection of electric pump to mains supply, check that the voltage stated on the plate correspond with the available mains supply.
- Do not use the electric pump in humid environments or close to water, also never touch the power cable with wet hands (risk of electric shock).
- After inflating/ deflating the mattress, switch off the inflater immediately to avoid overheating the motor.
- Do not use the inflater with adapters others than the one supplied with the product as they could damage the device.
- Do not inflate beyond max. air pressure, stated in technical data. This cannot happen, when using original electric pump, supplied by Airfun d.o.o.

## 7. CLEANING AND MAINTANANCE

### 7.1. Cleaning the upholstery:

- Clean the upholstery with a damp cloth (use lukewarm water and a microfiber cloth).
- Remove stubborn marks with a pH-neutral soap
- Never use agents that contain aggressive cleaners (strong detergents, solvents, bleaching or scouring agents).

### 7.2. Disinfecting the upholstery:

- Clean the upholstery first as described under point 7.1. before disinfecting.
- Follow the disinfectant's instructions for use.
- You can use disinfectants with light alcohol concentration or use Nubis disinfectant, recommended from Airfun d.o.o.

### 7.3. Cleaning of the metal parts:

- Clean metal parts of frame and legs with a soft dry cloth.
- In case of stubborn marks, a damp cloth and PH-neutral soap can be used.

### 7.4. User maintenance:

- Regular cleaning and disinfecting of the upholstery.
- We recommend to lubricate rotating part of the frame occasionally, at least once per year or earlier if function become troublesome.
- Anyway we recommend to regularly monitor main functions of the product and in case of any malfunctions contact authorized distributor or manufacturer.

## 8. TECHNICAL DATA

Product		NubisPRO 2.0	NubisPRO OSTEO 2.0	NubisPRO XL 2.0
<b>Code</b>		200-7520	300-6520	400-7520
<b>Mattress</b>	Dimensions (L x W x Thickness)	190 x 75 x20 cm	190 x 65 x 20 cm	210 x 75 x 20 cm
	Base/ cover material	DWF / PU coating	✓	✓
	Head section adjustment	0 - 15 cm (in length)	✓	✓
<b>Table</b>	Base material	Powder coated aluminium	✓	✓
	Height adjustment	58 - 80 cm	✓	✓
	Weight capacity	300 kg	✓	✓
	Table weight	10,9 kg	9,9 kg	11,9 kg
<b>Pump</b>	Power supply	230V~50Hz	✓	✓
	Max power	1000W	✓	✓
	Max pressure	250/3.6 mbar/psi	✓	✓
	Insulation safety class	Class II (double insulated)	 ✓	✓
<b>Backpack</b>	Dimensions	70 x 45 x 20 cm	✓	✓
	Transport weight (full backpack)	15,9 kg	14,9 kg	16,9 kg

Airfun d.o.o. reserves the right to change technical specifications.

## 9. ACCESSORIES AND SPARE PARTS

- 9.1. For the information on Nubis accessories, please refer to our current catalogues.
- 9.2. If spare parts or any other help is needed, please contact authorized distributor or Airfun d.o.o. for further assistance.

## 10. WARRANTY AND LIABILITY

### 10.1. Warranty:

- The warranty period for NubisPro is:
  - 2 years for the table base,
  - 1 year for table upholstery and other parts.The warranty starts with the day of Purchase.
- Should any repairs be necessary during warranty period, the customer must inform Airfun d.o.o. or your local dealer. The warranty must be verified by purchase invoice.
- The warranty obligates Airfun d.o.o. to carry the costs of repair of components which might fail during normal use inside the warranty period.
- Warranty is not applicable for defects or poor performance in following cases:
  - In the event of customer's improper use or neglect of NubisPro;
  - If customer doesn't follow requirements and recommendations of this User manual;
  - If any modifications or repairs are carried out by non-qualified service personnel, unauthorized by the manufacturer.

### 10.2. Product life expectancy and repairs:

- This product's expected life is estimated to be 5 years.
- Repair and spare parts are available during life expectancy, so 5 years from the date of purchase.
- Any repairs must always be performed by Airfun d.o.o. or any of its authorized technicians from local dealers.

### 10.3. Product liability:

The manufacturer or its suppliers or resellers cannot be liable for any consequences incurred by the user, the patient or the product itself as a result of improper use of the product, wrong interpretation or non-observance of this user manual, repairs carried out by unauthorized personnel.

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