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ROLL-ON PACK

Available MoVeS Roll-On Packs:

Product	Inside Ø	Circumference	Uses:
Finger	2,5cm	One size fits all	Finger
Small	10,0cm	Fits up to 25cm	Hand, Wrist, Arm, Elbow
Medium	12,5cm	Fits 25cm - 38cm	Hand, Wrist, Arm, Elbow, Ankle
Large	15,0cm	Fits 38cm - 53cm	Elbow, Ankle, Foot, Calf, Knee
Extra Large	17,5cm	Fits 53cm - 58cm	Knee, Thigh

DEFROST
1 minute
(+45 sec intervals)

75°C / 170°F
2 minutes

-18°C / 0°F
≥ 2 hours

WWW.MOVES-YOU.COM

Always read manual before use

Finger (not microwavable)

Small - Hand / Wrist

Medium - Arm / Elbow

Large - Ankle / Knee

Extra Large - Knee / Thigh

ENGLISH - Instructions for Use

Place insulating compression sleeve or equivalent (cloth or towel) over limb or treatment area. Gently roll the MoVeS Roll-On Pack over the limb to the desired position. Two fingers should easily slide between pack and treatment area. Remove the pack after 20 minutes or if it becomes uncomfortable. Wait 20 minutes prior to second application if required. Periodically check treatment area for irritation.

COLD Applications

Place MoVeS Roll-On Pack in freezer or refrigerator for two or more hours prior to use. Remove and gently knead to soften before use.

HOT Applications

- Hot water heating method (recommended): Place MoVeS Roll-On Pack in about 3 liter of hot tap water heated to desired temperature. Remove, test and if it feels too hot let cool. Dry before use. Maximum recommended water temp is 75°C / 170°F.

- Microwave Heating Method: Lay MoVeS Roll-On Pack flat in center of microwave and spread gel evenly. Heat on DEFROST for about 1 minute, remove, knead and test temperature. If required MoVeS Roll-On Pack may be heated on DEFROST additionally for intervals of 45 seconds. Knead and test temperature before use. Let cool if MoVeS Roll-On Pack feels too hot.

- Warning: Do not heat on high, overheating may occur. Do not heat finger pack in microwave.

Safety

Do not place MoVeS Roll-On Pack directly on skin. Never apply a MoVeS Roll-On Pack that is too tight, too hot or too cold. Keep out of reach of children. MoVeS Roll-On Pack is not a toy. Do not sleep with MoVeS Roll-On Pack. MoVeS Roll-On Pack is for external use only. MoVeS Roll-On Pack is a medical device. Do not sit, stand on or otherwise excessively compress or stretch as rupture may occur. Suggested action for contact with gel: thoroughly flush area with water & mild soap.

Storage and Handling

Freezer storage is recommended. To store and protect MoVeS Roll-On Pack out of the freezer for an extended period of time (more than a day) place in a heavy duty zip lock type freezer bag, force out air and seal closed. Return to freezer as soon as possible.

Cleaning

Hand wash, warm water with antimicrobial or regular hand soap, towel dry.

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Sport-Tec GmbH