DE 

EN 

FR 

IT 

ES 

NL 

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DESTIX DISINFECTANT WIPES

Art.-Nr.  28586 · Kategorie:  Präparate

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---

## WELCOME TO SPORT-TEC

Thank you for your purchase. We are pleased that you have opted for a Sport-Tec product and hope that you really enjoy using it. In this manual, you will find all the important information for use. If you have any further questions, please contact us – we will be happy to help you.

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---

## EN - MANUAL

Read this manual carefully before use. Follow the instructions, and keep this user manual in a safe place.

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## GENERAL INFORMATION

DESTIX is a unique disinfectant which is universally usable.

- effective against all bacteria, viruses, yeasts and fungi
- reliably prophylactic against the formation of resistance.

## USE BIOCIDES SAFELY.

**ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE.**

### DANGER:

Highly flammable liquid and vapour. Causes serious eye damage. May cause drowsiness and dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke. Avoid inhalation of vapour. Wear protective gloves. Get medical advice/attention if you feel unwell.

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove any contact lenses if possible. Continue rinsing. If eye irritation persists: Seek medical advice / attention.

Store in a closed container. Repeated contact may lead to chapped or cracked skin.

## CLEANING INSTRUCTIONS

DESTIX disinfectant wipes are ready-to-use wipes for rapid disinfection of surfaces and equipment in doctors' surgeries, care homes and clinics (e.g. treatment chairs, handpieces and contra-angle handpieces).

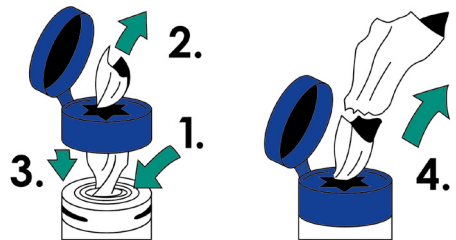
The wipes are not suitable for disinfecting invasive medical devices (e.g. instruments, hearing aids, ear moulds). Effective against bacteria, fungi and limited virucidal PLUS.

**Exposure time:** According to EN test standards: 1 min at high load.

**Application:** Only use on alcohol-resistant and non-porous surfaces. Wipe the surface thoroughly with a damp disinfectant wipe. Allow the liquid to act for 1 minute. Rub any residue on the surface dry with a clean, dry cloth. The cloth is sufficient for an area of 30 x 30 cm.

## NOTES ON USE

Open the lid of the tub, and cut open the bag, leaving the tissues inside the pouch. Pull the tissue from the middle through the distribution hole, then close the lid. For easy single tissue use, pull at an angle before tearing, then close the lid after use.

# CONTENTS

## DESTIX MA61

100 g disinfectant liquid contains: Ethanol: 24 g, 1-propanol: 37 g

## EXTRACT OF THE MICROORGANISMS TESTED

Microorganism	Contact time
Staphylococcus aureus + MRSA	1 Min
Escherichia coli	1 Min
Enterococcus hirae	1 Min
Pseudomonas aeruginosa	1 Min
Candida albicans	1 Min
Aspergillus niger	1 Min
Mycobacterium terrae	1 Min
Mycobacterium avium	1 Min
BVDV	1 Min
Vaccina	1 Min
Nora	1 Min
Rota	1 Min
Herpes Simplex Virus	5 Min
Adenovirus	5 Min

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**Sport-Tec GmbH**